


Small D = 18 mm  
REF 281481  
PZN 17373455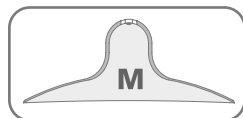


Medium D = 22 mm  
REF 281482  
PZN 17373461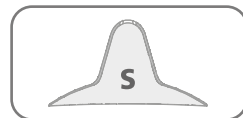



Small D = 18 mm  
REF 281441  
PZN 17373389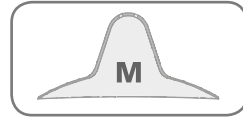

Medium D = 20 mm  
REF 281448  
PZN 17373403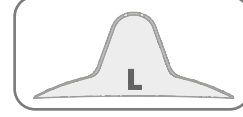

Large D = 28 mm  
REF 281442  
PZN 17373426

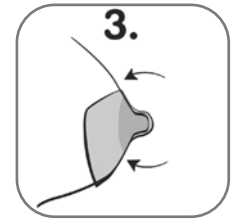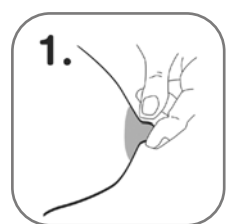KaWeCo GmbH  
Gerlinger Straße 36-38  
71254 Ditzingen  
Germany  
Tel.: +49 (0) 71 56 / 1 76 02 - 100  
Fax.: +49 (0) 71 56 / 1 76 02 - 500  
Email: info@kaweco.de

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## EN INSTRUCTION MANUAL Nipple shields

### 1. Purpose Why using mamivac® nipple shields?

Many mothers use conventional round shaped nipple shields when having sensitive and sore nipples. These should be used for a short time only, because they have influence on the natural way of breastfeeding. The round shape covers mum's skin, and it may happen that the baby refuses to suck when mum is trying to breastfeed again without a nipple shield. This is because the baby does not identify its mother due to the irritation by the nipple shield. The special shape of **mamivac®** nipple shields avoids this by reason of not interrupting the contact between mother and baby. Therefore the baby reminds the natural smell of its mother.

### 2. Use

Clean your breast before using **mamivac®** nipple shields. A short massage has the effect that nipples stand upright and make the use of **mamivac®** nipple shields much easier. Turn the soft edges up and press the **mamivac®** nipple shield as close as possible to your nipple. Press the soft edges afterwards to your skin. Wetting the **mamivac®** nipple shield supports the attachment to your skin. Make sure to place the **mamivac®** nipple shield with the flat side up that your baby's nose has direct contact to your skin while breastfeeding. Then the baby identifies your smell and feels safe. Do not use Vaseline or other cremes. For further information regarding the use please ask your lactation consultant or midwife. Prior to using the nipple shields, check them for cracks and proper functionality.

### 3. Cleaning and disinfection

**mamivac®** nipple shields consist of thin, transparent and tasteless silicone. Prior to the first use and before any other use, disinfect the nipple shields by boiling them in water for at least 5 minutes. Clean the nipple shields after each use with commercial washing-up liquid and rinse them thoroughly with lukewarm water. With proper use and storage, the **mamivac®** nipple shields have a service life of 2 years. Keep the operating instructions. This product is not a toy. Keep it away from children.

### 4. Disposal

Please dispose of your **mamivac®** nipple shields in accordance with your local disposal regulations for plastic products.

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