**mamivac®**

# **HEBAMMENSTETHOSKOP PINARD STETHOSCOPE**

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Operating instructions

Stand / As of 2021-10  
GB2050002M

**mamivac®.com**  
  
**Made in Germany**

 KaWeCo GmbH  
Gerlinger Str. 36-38  
71254 Ditzingen  
Germany

Tel.: +49 (0) 71 56 / 1 76 02 - 100  
Fax: +49 (0) 71 56 / 1 76 02 - 500  
info@kaweco.de

**mamivac®.com**  
  
**Made in Germany**

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Read these instructions carefully before using the pinard stethoscope and keep them in a safe place for future reference.

**Intended use:**

The Pinard stethoscope is a diagnostics means to assess acoustic phenomena in the body, in particular fetal and maternal heart sounds.

**Contraindication/side effect:**

None known.

**Patient target group:**

Pregnant women.

**Users:**

This medical device should only be used by trained personnel.

**Application:**

Check the product for any visible damage and functionality prior to each use. In the event of defects or malfunctions, discontinue use of the product.

Place the flat side of the pinard on your ear.

**Material:**

The stethoscope for midwives is made from natural, untreated beechwood. Each Pinard is a one-off, being unique in grain and colouring. The hand-turned stethoscope is coated with solventfree oil-wax.

**Cleaning:**

Wipe off the pinard stethoscope with a lint-free cloth damped with a pH neutral detergent. Product is not suited for automated cleaning procedures.

**Disinfecting:**

In general, it will be sufficient to disinfect the surfaces which come into contact with patients with a disinfecting wipe after each use. Use alcohol-based disinfectants approved by the German Society of Hygiene and Microbiology DGHM. Do not use automated disinfection methods.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the national competent authority.

REF 2260

kurz / short 18 cm

REF 2261

lang / long 33 cm

Medical device

Conformity mark according to Regulation (EU) 2017/745