mamivac®

LACTIVE

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mamivac® LACTIVE

Operating Instructions – electric breast pump
For use in the domestic environment

CE 0124

REF 287000
As of: 2022-02
GB1030009GB

mamivac.com
■ ■ ■ ■ ■
Made in Germany

Please read the operating instructions carefully before you use the **mamivac® LACTIVE** breast pump. Keep the instructions for later reference.

Contents

1.	Proper use	17
1.1	Indications	17
1.2	Contraindications	17
1.3	Side effects	18
2.	Assembling and components	18
3.	Using the breast pump	19
3.1	The stimulation phase	19
3.2	The expressing phase	19
3.3	Switch off the pump	20
4.	Hygiene and cleaning	20
4.1	Cleaning and servicing the breast pump	20
4.2	Cleaning and servicing the suction kit	20
5.	Important safety notes	20
5.1	Note on electromagnetic compatibility (EMC)	21
6.	Storage, transportation and ambient conditions	21
7.	Warranty	21
8.	Practical tips	22
8.1	General notes	22
8.2	Storing breast milk	22
9.	Errors and solutions	23
10.	Environmental protection and disposal	23
11.	Technical data	23
12.	Conversion table of units	24
13.	Legend	24
14.	Spare parts and accessories	25

Congratulations!

Thank you very much for buying our product. We wish you and your baby good luck and health and a pleasant breastfeeding time in which we want to support you to the best with our **mamivac® nursing products**. For other **mamivac®** breastfeeding products, please refer to our website **www.mamivac.com**.

1. Proper use

With the **mamivac® LACTIVE**, you can express and collect breast milk gently in case of breastfeeding problems. Breastfeeding problems may be caused by the mother herself or the baby.

The electric breast pump **mamivac® LACTIVE** can be used to

- increase milk production by completely emptying the breast.
- maintain the breastfeeding option for the baby to continue normal breastfeeding after breastfeeding problems.

1.1 Indications

The **mamivac® LACTIVE** breast pump is used to gently express and collect milk in case of breastfeeding problems or other situations where direct breastfeeding is not possible. Medical indications for using the **mamivac® LACTIVE** breast pump are the following breastfeeding problems:

- mastitis (bacterial inflammation of the mammary glands)
- sore mamillae (nipples)
- anatomic variants of the mamillae (flat or inverted nipples) which make feeding the baby more difficult
- milk oversupply (excessive milk production, poorly sucking child)
- insufficient milk production
- premature babies (poorly sucking premature infant or separation of infant from mother)
- metabolic disorders of the baby
- anatomic disorder in the face of the infant (cleft lip, jaw, palate)
- diseases of the mother where breastfeeding is temporarily contraindicated (e.g. taking medicine).

1.2 Contraindications

A possible contraindication is being allergic to the silicone of the breast shield which is in direct contact with the skin. The breast shield is part of a few **mamivac® suction kits**. When taking specific medicine, breastfeeding the infant is also contraindicated since pharmacologically effective substances can pass through the milk. For information about possible contraindications for breastfeeding, refer to the Patient Information Leaflet provided by the pharmaceutical company. When medication is limited to a specific period of time, expressing the milk with a breast pump helps to maintain milk production over this period.


Do not use mamivac® pumps:

- to suction other body fluids, such as blood, secretions or mucus.
- to suction outside the medical field, such as corrosive, flammable or explosive liquids.
- on other living beings.

1.3 Side effects

- In individual cases, the skin may be irritated resulting in redness or red spots.
- In case of a known silicone allergy, do not use the breast shield made of silicone. The breast shield is part of a few **mamivac® suction kits**.

2. Assembling and components mamivac® LACTIVE (REF 281024)

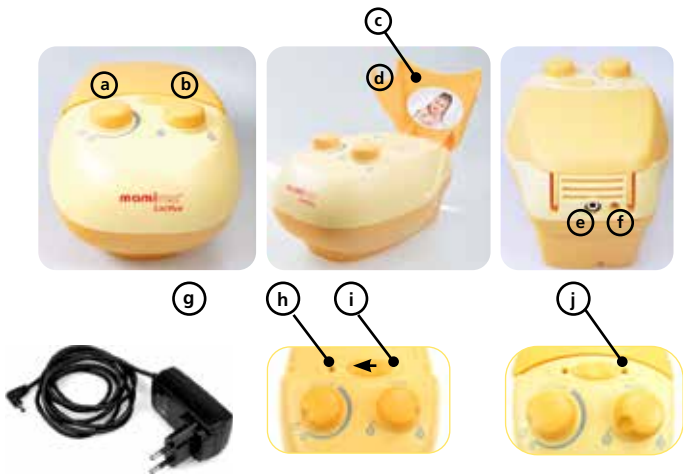- a) Continuously adjustable vacuum control (Vacuum)
- b) 2-phase cycle control (Phase)
- c) Stick a photo of your baby here to give yourself a visual stimulus to support the let-down reflex.
- d) Tiltable lid with lateral mount for suction kits
- e) Socket for power supply unit
- f) LED operating hour indicator (green)
- g) Power supply unit (figure shows EU version)
- h) Socket for suction unit
- i) Sliding lid for one or two suction kits
- j) Socket for second suction unit

3. Using the breast pump

Prior to using the pump, please observe the following:


- To avoid malfunction and ensure a proper functioning, use the original **mamivac® suction kits** exclusively.
- Please observe all cleaning and disinfection notices in Chapter 4.
- Assemble the parts as shown in the operating instructions.
- Please use air hoses with the following dimensions only: 5.5 mm outside Ø, 2.5 mm inside Ø, about 1 m long.

Adjusting suction strength and cycle speed with the **mamivac® 2-phase technology**

Our **mamivac® LACTIVE** breast pump simulates the baby's natural sucking behaviour. It can be divided into two phases (stimulation phase and expressing phase).

3.1 The stimulation phase

Phase 1 and low vacuum: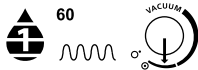


Imitates the baby's sucking which is fast, but weak at the beginning. The short cycles stimulate the flow of milk. Perform the following steps:

Switch on the pump by turning the control to the mark .

Turn the phase control to . This corresponds to a cycle speed of about 60 cycles per minute. Make sure that the vacuum control is in the first third of the vacuum strength display (low vacuum).

Let-down reflex

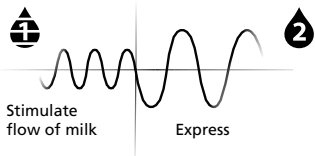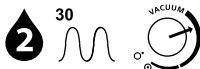
When the milk starts to flow, switch to the expressing phase, after two minutes at the latest.

3.2 The expressing phase

Phase 2 and increased vacuum:

If the flow has been stimulated, the milk starts to flow. Now, you can express the milk very gently and efficiently.

Turn the phase control to . This corresponds to a cycle speed of about 30 cycles per minute. In the expressing phase, you can adjust the vacuum strength steplessly.

3.3 Switch off the pump

Turn the vacuum control to the position . The device is now switched off.

4. Hygiene and cleaning

4.1 Cleaning and servicing the breast pump

- When the pump is not in use or when it is cleaned, disconnect it from mains supply. In case of problems, also disconnect the pump immediately by all means.
- Do not use aggressive detergents (solvents, etc.) to clean the casing but a cloth dampened with a light soap sud. Wipe over the parts with a dry cloth. Make sure that no liquid enters the device.
- For the disinfection of the casing, you can use alcohol-based disinfecting agents.

Do not rinse the pump!

4.2 Cleaning and servicing the suction kit

- For cleaning the suction kit, refer to the operating instructions of the respective suction kit.
- Prior to expressing milk, clean all parts of the kit which come into direct contact with breast milk.
- To disinfect the parts for private use we recommend to boil the parts in a pot (no pressure cooker) for at least 15 minutes. You can also use a steriliser. Do not use chemical disinfection baths.

Prior to assembling the suction kit, make sure that the diaphragm is completely dry and undamaged (no cracks, holes, etc.).

5. Important safety notes

- Make sure that the supplied voltage matches the voltage stated on the power unit or is within the limits.
- Do not operate the **mamivac® LACTIVE** in operating rooms or in rooms with explosion risk.
- The breast pump is not designed to be used outdoors or during transportation.
- The breast pump is intended for use at home.
- Operate the **mamivac® LACTIVE** breast pump on a flat and stable surface.
- Operate and store the **mamivac® LACTIVE** breast pump in dry rooms only.
- In case of malfunctions, please contact the manufacturer or your sales representative.
- If you think a safe operation is no longer ensured, e.g. in case of damaged insulation of the power cord or damaged cord between the power unit and the device, discontinue the operation of the device immediately (unplug power cord from wall outlet) and secure it against putting into operation again. Unplugging is the only means to safely disconnect the device from the public power supply.
- Any modifications/manipulations of the device or its accessories, or a combination with other devices and/or systems or components make a safe use impossible and are therefore strictly forbidden.

- Prior to using the device, check the milk bottle for cracks and exchange it if need be. A frequent boiling of the milk bottle and the parts of the suction kit may cause a brittling of the plastic material resulting in cracks. Porous parts shall be replaced by new parts.
- Small parts are enclosed to the product which could be inhaled or swallowed. Keep them away from toddlers/children

5.1 Note on electromagnetic compatibility (EMC)

The **mamivac® LACTIVE** breast pump described in these instructions for use may only be operated in accordance with the following information, provided that the accessories and spare parts mentioned in chapters 11 and 14 are used, in an electromagnetic environment that corresponds to a domestic environment (dwelling, medical practices) where the public power system is available. It is not permitted to operate the **mamivac® LACTIVE** breast pump in means of transportation, be it over land, across water or through the air, nor in special-use HF shielded rooms (e.g. MRT) where electromagnetic bursts of high intensity may occur nor in the immediate vicinity of active HF surgery devices or shortwave therapy devices.

A detailed technical description of the electromagnetic environment can be found in the enclosed EMC guidelines.

Provided that the **mamivac® LACTIVE** breast pump is operated in the specified environment, an impairment of the performance characteristics given in chapter 11 or the basic safety is not to be expected.

WARNING: Avoid using this device directly next to other devices or with other devices stapled on top of each other as this could disturb operation.

WARNING: The use of accessories, converters or cables other than those defined or provided by Kimetec for this device can lead to an increased electromagnetic interference or a reduced electromagnetic immunity of the device resulting in faulty operation.

WARNING: Portable HF communication devices (transmitters), including their accessories such as antenna cables and external antennas, shall not be used within a distance shorter than 12 inches (30 cm) to the parts and cables listed by Kimetec for the **mamivac® LACTIVE** breast pump. Non-observance may impair the performance characteristics of the device.

6. Storage, transportation and ambient conditions

- Avoid strokes or impacts of any kind, regardless of whether or not the device is packed.
- Avoid rapid temperature changes, high air humidity or a dusty environment.
- The device can be operated at an ambient temperature between +5°C and +40°C, a relative humidity between 15 % and 93 % (non-condensating), and air pressure between 700 hPa and 1060 hPa.
- Store and transport the device at an ambient temperature of -29°C and +60°C, a relative humidity of max. 85%, and air pressure between 500 hPa and 1060 hPa.
- An excessive accumulation of lint and dust in the inside of the device impairs heat dissipation of components, which may overheat as a result representing a fire hazard.

7. Warranty

- The warranty covers 12 months from date of purchase, yet a maximum of 300 operating hours, if these are reached prior to the expiry of 12 months. The green operating hour indicator goes out when 300 operating hours are reached. The suction kit is exempted from the warranty.
- **Disclaimer of warranty:** Improper use, opening the device or using suction kits not specified for this breast pump will void any warranty claim.

8. Practical tips

8.1 General notes

- Prior to using your suction kit for the first time and after every usage, clean the suction kit and assemble it as described in the respective operating instructions.
- Prior to expressing, wash your hands thoroughly with soap and wipe the breast with a damp cloth.
- To avoid dry, sore or cracked nipples, do not use soap or alcohol as detergent.
- Prepare yourself for the expressing with ease. Take a comfortable and relaxed seat.
- Gently press the breast tube over the nipple for a tight contact.
- You can operate the **mamivac® LACTIVE** breast pump either with one or two suction kits. To save time, it is possible to express both breasts simultaneously (**double pumping**).
- Start to express milk. Please observe the instructions for operating explained in Chapter 3.
- The pump has a stimulation mode to support milk production. You can also use your free hand for an additional breast massage to support the flow of milk.
- Express from both breasts alternately, unless instructed otherwise by the physician.
- Since expressing times strongly depend on the individual mother, you are asked to contact your midwife or breastfeeding assistant.
- As a rule of thumb, the following expressing times are recommended:
With one suction kit: Expressing scheme: Express first and second breast 5 minutes each one after the other, followed by 3 minutes and 2 minutes each.
With two suction kits: Express both breasts simultaneously for 10 minutes (**double pumping**). Have a look at the fill level of the bottle at the suction kit from time to time and make sure it does not overflow.

8.2 Storing breast milk

Breast milk can be stored as follows:

- Freshly expressed milk at room temperature (16° to 29°C) 3-4 hours
- In the rear of the fridge (below 4°C) 3 days
- In the freezer (-18°C) 6 months

Fill the used bottles or containers to two thirds only and write down the fill date. Instead of a bottle, you can also use our specially designed **mamivac® breast milk freezer bags** [REF 282100]. Do not defrost frozen breast milk in the microwave, but at room temperature. Place the bottle or bag in warm water as required. After defrosting, gently shake the container or bag to mix the milk.

9. Errors and solutions

The protective diaphragm effectively prevents that liquids or solids are sucked into the pump. If substances are sucked in nevertheless or any other problem occurs, immediately put the device out of operation. Contact your dealer or the customer service of your shop.

Problem	Error/solution
Pump does not start	<ul style="list-style-type: none">- Check connections of power supply.- LED on power unit must light (if existing).- If the problem is not solved, please contact the customer service of your shop.
Pump does not suck (does not build up vacuum)	<ul style="list-style-type: none">- Switch off and on the device several times. Wait a minute before switching on again.- Check the respective suction kit for proper assembly.- Check tube valve (for cracks and dirt).- Check protective diaphragm for cracks.- Protective diaphragm must be inserted flush and crinkle free.- Check hose connections.

The breast pump/power unit must not be opened by users/patients. The device is maintenance-free. For repair work, the device must be returned to the manufacturer.

10. Environmental protection and disposal

Please dispose of a disused **mamivac® LACTIVE** breast pump in accordance with your local disposal regulations for electrical devices.

11. Technical data

The expected service life is 2 years.

mamivac LACTIVE

Nominal voltage:	12 V DC
Power consumption:	1.00 A
H x W x D (mm):	115 x 125 x 195
Weight:	630 g
Max. vacuum:	-33 kPa (\pm 15%)
Mean vacuum:	Device for intermittent sucking without continuous vacuum

REF 281024

Device classification:

Classified acc. to 93/42/EEC:
Article 9, Appendix IX:
Class IIa/Rule 11

Power supply unit

Nominal input voltage:	100-240 V AC 50 / 60 Hz
Input current:	500 mA
Output voltage:	12 V DC
Output current:	1,25 A
Protection class:	II
H x W x D (mm):	75 x 46 x 30,5
Weight:	120 g

REF 283005

EU, UK, US version

Power supply unit**REF 283021****EU version**

(exchangeable plug-in adapters available on request)

Nominal input voltage: 100-240 V AC 50 / 60 Hz

Input current: 400-200 mA

Output voltage: 12 V DC

Output current: 1500 mA

Protection class: II

H x W x D (mm): 95 x 42 x 67

Weight: 157 g

12. Conversion table of units**1 kPa**

corresponds:

0.01 bar

0.14504 PSI

corresponds:

7.5006 mmHg

(pounds per square inch)

(millimetre mercury, at 0°C)

13. Legend

	Please observe safety instructions		Transportation and storage: -29°C to +60°C Operation: +5°C to +40°C
	Observe operating instructions		Manufacturer
 0124	Conformity mark according to EU Council Directive 93/42/EEC	IP41	Protection against penetration of solid particles of ≥ 1.0 mm diameter and vertically falling water drops.
	Device of protection class II		Direct current
	Component of type BF		Alternating current
	On		Input current
	Off		Output current
	Do not dispose of in domestic waste	SN	Serial number
REF	Item no.		Store in dry places
	Observe operating instructions		Protect from sunlight

14. Spare parts and accessories

Designation		REF
mamivac® Soft bag (red)		283014
mamivac® Suction kit PREMIUM S		283129
mamivac® Suction kit PREMIUM L	with soft insert and hygienic cover	283120
mamivac® Double suction kit PREMIUM L	- -	283123
mamivac® Suction kit PREMIUM-light L	without soft insert and hygienic cover	283120K06
mamivac® Suction kit PREMIUM XL		283133
mamivac® Breast shield S	Ø 20, mm	283131
mamivac® Breast shield L	Ø 28 mm	283132
mamivac® Breast shield XL	Ø 36 mm	283121
mamivac® Suction kit ECONOMY L	small bottle, without soft insert and hygienic cover	283122
mamivac® Suction kit ECONOMY XL	- -	283128
mamivac® Double suction kit ECONOMY L	- -	283124
Power supply unit EU, US, UK version		283005
Power supply unit EU version		283021
Operating instructions		GB1030009M

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