# mamivac®

## BABYFEED

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#### Vorteile:

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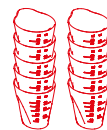
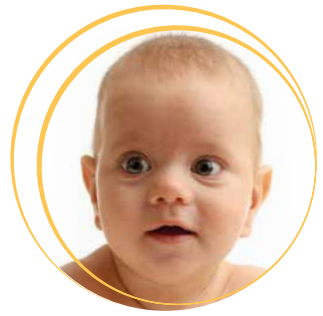
### Infant feeding helper

#### Advantages:

- Quick and easy use
- Feeding possibility when mother and baby are separated
- Does not cause nipple confusion
- Does not satisfy the baby's desire for sucking, you can breastfeed afterwards
- Allows the additional feeding of the hindmilk with its higher fat content
- Baby controls quantity and speed of nursing (in contrast to bottle feeding)

#### Instructions for use:

When feeding with the cup, wrap the awake baby into a towel to prevent him or her from slapping away the cup. Bring the cup to the baby's mouth in a way that the rim slightly touches the bottom lip. Now, incline the cup to the degree that milk just touches the baby's lips for him or her to become active. Premature babies tend to lick like a cat while mature infants are likely to sip and slurp (use bib or napkin). Like with breastfeeding each baby will develop his or her own style and need for breaks.

**Before the first and after any use, clean and disinfect (sterilize) the cup.**

#### Cleaning and sterilizing:

You can use soap suds or mild alkaline detergents for cleaning. For disinfection, use commercial instrument disinfection which are suited for plastics. It is mandatory to observe the manufacturer's specifications regarding reaction time and applied concentration. Disinfectants containing phenol derivatives or alkylamines as agents are not suited. Can also be disinfected by boiling in water for at least 5 minutes.

#### Sterilizing:

The polypropylene Babyfeed is autoclavable up to 121 °C. A sterilization method by moist heat validated in accordance with EN 554 can be used for sterilizing. It is recommended to recondition the cups not more than 100 times. Carefully check the cup for any damage before using them.

REF 282112  
Content: 2 pieces

REF 282113  
Content: 10 pieces