# mamivac® EASY

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# mamivac® EASY

As of: 2024-06  
GB1020001GB

For expressing breast milk in case of breastfeeding problems,  
illness or separation of mother and infant.

Observe safety instructions

Transportation and storage: -25°C to +70°C  
Operation: +5°C to +40°C

Observe operating instructions

Protect from sunlight

Medical device

Manufacturer

Unique Device Identifier

Kimetec GmbH  
Gerlinger Str. 36-38  
71254 Ditzingen  
Germany

Phone: +49 (0) 71 56 / 1 76 02 - 200  
Fax: +49 (0) 71 56 / 1 76 02 - 500  
info@kimetec.de

mamivac.com  
  
Made in Germany

Before using the **mamivac® EASY** manual breast pump, read the operating instructions carefully and keep them for future reference.

## Inhalt

1.	Notes on the operating instructions	12
2.	Assembling and components	13
3.	Purpose	14
3.1	Indication	14
3.2	Contraindication	14
3.3	Side effects	14
3.4	Target group of patients/users	14
4.	General notes	14
4.1	Functioning of the manual breast pump	15
5.	Hygiene and cleaning	16
6.	Practical tips	16
6.1	Storing breast milk	16
6.2	Accessories	17
7.	Classification	17
8.	Disposal	17
9.	Spare parts	17
10.	Warranty	17
11.	Technical data	17

## 1. Notes on the operating instructions

These operating instructions are valid for the **mamivac® EASY** manual breast pump as well as all other derivatives of identical design.

## 2. Assembling and components

a) Pump unit with vacuum control

b) Hygienic cover

c) Soft insert

d) Breast shield


e) Silicone valve

f) Bottle

g) O-ring seal

h) Bottle cap

i) Piston seal ring

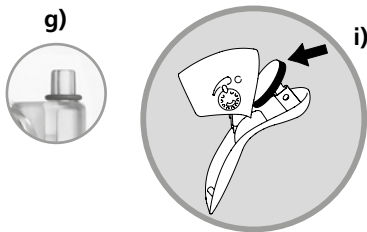### 3. Purpose

The manual breast pump **mamivac® EASY** is used in order to gently and softly express and collect breast milk.

#### 3.1 Indication

Breastfeeding problems that require the use of a breast pump may be: mastitis (inflammation of mammary glands), aching or sore mamillae (nipples), anatomic variants of the mamillae which make feeding the infant difficult (flat or inverted nipples), insufficient or excessive milk production, premature delivery, metabolic disorders of the infant, anatomic disorder in the face of the infant, poorly sucking infant, temporary illness of the mother (medication intake).

#### 3.2 Contraindication

Taking medication in general may affect breastfeeding an infant. The patient information leaflets of the pharmaceutical companies contain appropriate information.

**mamivac® EASY may not be used:**

- to suction other body fluids, such as blood, secretions or mucus
- to suction outside the medical field
- on other living beings.

#### 3.3 Side effects

In individual cases, the skin may be irritated resulting in redness or red spots.

#### 3.4 Target group of patients/users

Breastfeeding mothers

### 4. General notes


- Prior to using the product, always check it for external damage and proper working. Do not use the product in case of faults or faulty functions.
- Prior to expressing, wash your hands thoroughly with soap and wipe your breast with a damp cloth. Do not use soap or alcohol as detergent to avoid dry, soar or cracked nipples.
- Prepare yourself for the expressing with ease. Take a comfortable and relaxed seat. Gently press the breast shield over the nipple for a tight contact. Start to express milk.

- The pump has a stimulation mode to trigger the let-down reflex. You can also use your free hand for an additional breast massage.
- Express from both breasts alternately, unless instructed otherwise by the doctor. Since expressing times strongly depend on the individual mother, you are asked to contact your midwife or lactation consultant.
- If you feel pain or experience any problems, contact your doctor or midwife/lactation consultant.
- Any severe incident that occurs in connection with the product shall be reported to the manufacturer and the responsible national authority.

## 4.1 Functioning of the manual breast pump

### The stimulation phase (1):

To trigger the let-down reflex, turn the vacuum control to the very left. Press the pump handle dynamically downwards several times. This imitates the baby's natural sucking behaviour. Keep the pump handle pressed for 3-4 seconds, so that the milk starts to flow into the bottle.

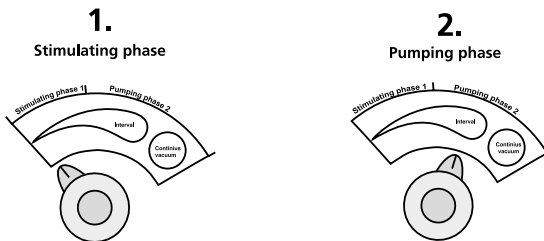### The expressing phase (2):

When the milk is flowing, turn the vacuum control to the right. You can express now by pressing the pump handle downwards and keeping the handle pressed until the vacuum has been released.

If the milk does not start to flow right away, relax and try again – even at a later point in time.

If expressing causes pain, do not use the pump anymore! Please contact your doctor or your midwife/lactation consultant. Make sure not to express too much milk into the bottle. Keep the pump in an upright position.

## 5. Hygiene and cleaning

Before you use the breast pump for the first time, clean components b) to h) and boil in boiling water for at least 5 min. or use a steam disinfection device, such as a steriliser for instance. Please observe the manufacturer's corresponding notes.

**Do not use a microwave oven.**

- Do not use chemical disinfection baths, solvents or abrasives.
- All components b) to h) can be cleaned with commercial washing-up liquids by hand or in the dish washer.
- Prior to using the pump the next time, clean the parts which came into contact with milk, and check them for proper functioning and condition.
- After cleaning, rinse components b) to h) thoroughly with lukewarm water to remove residues of the washing-up liquid, especially on those components which come into direct contact with breast milk. You might use a brush to clean areas which are difficult to reach.
- Assemble the manual breast pump according to the drawing in »Assembling and components«.
- In case the piston is sticky, apply talcum or baby powder to the piston seal ring (i).

## 6. Practical tips


Wrap the cleaned and dry parts in a clean cloth and store them in a container when not in use.

### 6.1 Storing breast milk

**Breast milk can be stored as follows:**

- Freshly expressed at room temperature (16-29°C) 3-4 hours
- In the rear of the fridge (below 4°C) 3 days
- In the freezer (-18°C) 6 months

Fill the used bottles or containers to two thirds only and write down the fill date. Instead of a bottle, you can also use our specially designed **mamivac®** breast milk freezer bags [REF 282100]. Do not defrost frozen breast milk in the microwave, but at room temperature. Place the bottle or bag in warm water (37°C) as required. After defrosting, gently shake the container or bag to mix the milk.

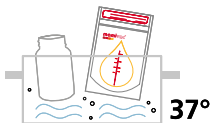## 6.2 Accessories

- Soft insert REF 281215
- **mamivac® breast milk freezer bags** PU = 20 pcs. REF 282100
- Hygienic cover REF 49010147

## 7. Classification

Medical product of risk class I

## 8. Disposal

Please dispose of your **mamivac® EASY** manual breast pump in accordance with your local disposal regulations.

## 9. Spare parts

Silicone valve	REF B49054105
Piston seal ring	REF 49054125
O-ring seal	REF 49054005
Hygienic cover	REF 49010147
Standard breast shield	REF 28012011
Bottle 120ml with cap	REF 28011131
Retaining spring	REF 49040100

## 10. Warranty

Ignoring the notes in these operating instructions or an improper use of the device will void any warranty claims. Kimetec GmbH rejects any liability for personal injuries and material damages arising from this.

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