mamivac®

THERMO-PACK

Stand 2022-02
GB1030012M

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mamivac®

■■■■ THERMO-PACK

Directions of use mamivac® Thermo-Pack

Observe directions of use.

Maximum heating +80°C
Maximum cooling -18°C

Kimetec GmbH
Gerlinger Str. 36-38
71254 Ditzingen
Germany

Tel.: +49 (0) 71 56 / 1 76 02 - 200
Fax: +49 (0) 71 56 / 1 76 02 - 500
info@kimetec.de

mamivac®.com
■■■■

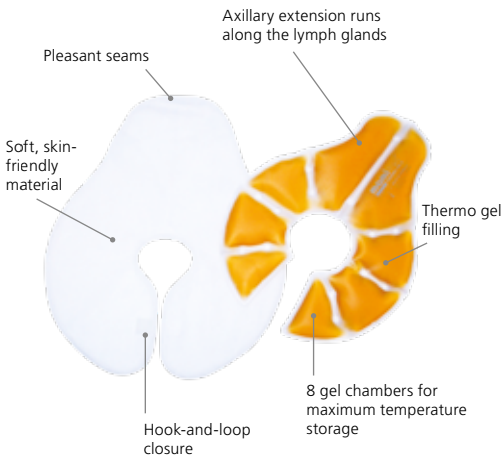
Made in Germany

1. Purpose

The **mamivac® Thermo-Pack** is an anatomically shaped gel cushion which can be used both for heating and cooling the breast:

The anatomically shaped design allows for a treatment of the lymph and mammary glands. Applied as gel cushion prior to breastfeeding, the **mamivac® Thermo-Pack** can support the flow of milk. Applied as cold cushion after breastfeeding, it soothes feelings of tension. The **mamivac® Thermo-Pack** can also be used as heat or cold cushion on many body parts (e.g. on a knee or in the neck).

2. Components and correct placing

Heating:

In hot water bath (approx. 60°C) for about 5-10 minutes

Cooling:

Place in the refrigerator for about an hour

Material:

The high-quality thermo-gel pad is temperature resistant from -18 °C to +80 °C. It is non-toxic.

Place the **mamivac® Thermo-Pack** wrapped in the **mamivac® protective fleece bag** on your breast to have the nipple in the opening. Make sure that the entire areola is within the opening. The long side of the **mamivac® Thermo-Pack** shall run along the lymph glands in the armpit (see figure).

3. Using

3.1 Warm cushion:

- Heat the **mamivac® Thermo-Pack** in a hot water bath at a maximum temperature of 80°C for about 10 minutes. Hot water from the tap (about 60°C) will also be absolutely sufficient.
- It is not allowed to heat the **mamivac® Thermo-Pack** in a microwave oven, since the material might become brittle or the Thermo-Pack may burst.
- Place the **mamivac® Thermo-Pack** wrapped in the protective bag on the breast about half an hour before breastfeeding or expressing milk.

3.2 Cold cushion:

- The **mamivac® Thermo-Pack** can be cooled in the freezing compartment (max. -18 °C). However, we recommend to cool the gel cushion in the refrigerator (about 1 hour) since great cold puts excessive stress on the skin.
- After breastfeeding or expressing milk, place the **mamivac® Thermo-Pack** wrapped in the protective bag on the breast for about 10 minutes.

4. Safety information

- Do not apply the **mamivac® Thermo-Pack** too hot or too cold. If the temperature of the breast gel cushion is too high or too low, you may suffer burns. Check the temperature with the back of your hand or the underside of your arm before you place the **mamivac® Thermo-Pack** on the breast or other body parts.
- To protect the skin against excessive heat or cold, always use the breast gel cushion covered in the protective bag.
- The protective bags have a hook-and-loop closure to adjust the gel cushion to the individual breast size.
- **Do not place on open wounds!**
- Do not heat the **mamivac® Thermo-Pack** above 80°C.
- Do not swallow the content in case of a damaged, leaking or cracked plastic shell.
- The content is non-toxic.
- Please dispose of leaking **mamivac® Thermo-Packs** immediately observing the legal regulations for domestic waste.
- Do not apply on infants.
- Keep the **mamivac® Thermo-Pack** out of reach of children.
- Only use cleaning and disinfecting agents which are suitable for soft PVC to clean the **mamivac® Thermo-Pack**.

- **Protective bags** 5 pcs. per unit
- **Thermo-Pack** incl. 3 protective bags
- **Soft, washable velvety bags**

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