# mamivac®

**BRUSTGLOCKE S, L, XL**  
**BREAST SHIELD S, L, XL**  
**CLOCHE NUE S, L, XL**  
**COPA DE SUCCIÓN S, L, XL**  
**CAMPANA PER MAMMELLA S, L, XL**  
**CONCHAS PARA O SEIO S, L, XL**  
**GÖĞÜS HUNİSİ S, L, XL**

**für Pumpset PREMIUM**  
**for suction kit PREMIUM**

Stand / As of 2021-09  
GB1010025M

CE 0124

**KaWeCo GmbH**

Gerlinger Str. 36-38  
71254 Ditzingen  
Germany

Tel.: +49 (0) 71 56 / 1 76 02 - 100  
Fax: +49 (0) 71 56 / 1 76 02 - 500  
info@kaweco.de

**mamivac®**

**Made in Germany**

DEUTSCH	1
ENGLISH	6
FRANÇAIS	11
ITALIANO	16
ESPAÑOL	21
PORTUGUÊS	26
TÜRKÇE	31

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mamivac®  
■■■■

Made in Germany

Before using the **mamivac® breast shield**, read the operating instructions carefully and keep them for future reference.

## Contents

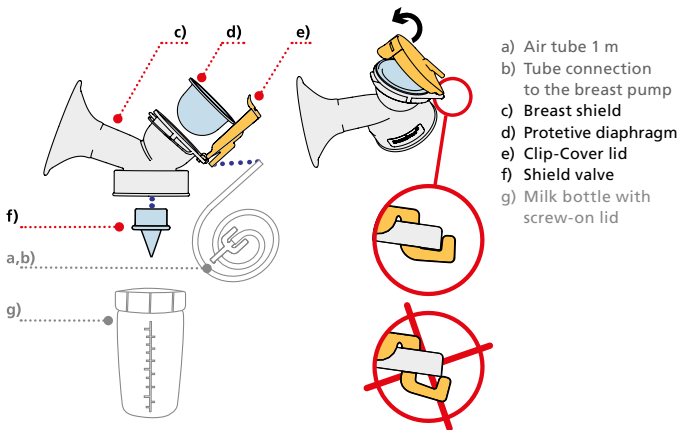
1.	Notes on the operating instructions	7
2.	Layout and components	8
3.	Purpose	8
3.1	Indication	8
3.2	Contraindications	8
4.	Hygiene and cleaning	9
5.	Disposal	9
6.	Spare parts	10
7.	Legend	10
8.	Warranty	10
9.	How to determine the correct breast shield size?	10

## 1. Notes on the operating instructions

These operating instructions are valid for all breast shields for **mamivac® suction kits PREMIUM** as well as all identically constructed models to be used with **mamivac® LACTIVE**, **mamivac® SENSITIVE-C** and **mamivac® SENSITIVE-CFH** breast pumps.

Breast shield S	REF 283131
Breast shield L	REF 283132
Breast shield XL	REF 283121

## 2. Layout and components

## 3. Purpose

### 3.1 Indication

For an efficient and comfortable expressing, it is indispensable to use a breast shield in the correct size. For this reason, **mamivac® breast shields** are available in the three sizes S, L and XL (20 mm, 28 mm, 36 mm) for the **mamivac® suction kit PREMIUM**.

The patented protective diaphragm in the breast shield offers the best possible protection and safety for mother and baby. This diaphragm hermetically shuts off the breast pump. Breast milk, bacteria or germs cannot contaminate neither the tubes nor the pump.

**Also observe the operating instructions of the breast pump.**

### 3.2 Contraindications

A possible contraindication is being allergic to the silicone of the optional soft insert which is in direct contact with the skin. In case of a known silicone allergy, do not use the silicone insert. In individual cases, the skin may be irritated resulting in redness or red spots.

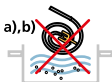
**mamivac® suction kits may not be used:**

- to suction other body fluids, such as blood, secretions or mucus
- to suction outside the medical field, such as corrosive, flammable or explosive liquids
- on other living things

## 4. Hygiene and cleaning

Before you use the suction kit for the first time, boil components c) to f) in boiling water for at least 15 min. or use a steam disinfection device, like a sterilizer for instance. Please observe the manufacturer's corresponding notes.

- Do not use chemical disinfection baths, solvents or abrasives.
- All parts of the suction kit, except for the air tube and the tube connection, can be cleaned with commercial washing-up liquids by hand or in the dish washer.
- Prior to using the kit the next time, clean the parts which came into contact with milk. It is not necessary to clean the air tube and the tube connection. It is sufficient to wipe off these parts with a damp cloth.
- After cleaning, rinse components c) to f) thoroughly with lukewarm water to remove residues of the washing-up liquid, especially on those components which come into direct contact with breast milk. You might use a brush to clean areas which are difficult to reach.

**Prior to assembling the pump, make sure that the diaphragm is completely dry and undamaged (no cracks, holes, etc.).**

- Assemble the suction kit according to the drawing in »Layout and components«.

## 5. Disposal

Please dispose of **mamivac® suction kits** in accordance with your local disposal regulations for plastic products.

## 6. Spare parts

Silicone protective diaphragm

REF 49054124

## 7. Legend

	Observe safety instructions.		Store in dry place
	Observe operating instructions.		Manufacturer
	Conformity mark according to EU Council Directive 93/42/EEC		Protect from sunlight

## 8. Warranty

Ignoring the notices in these operating instructions or an improper use of the device will void any warranty claims. KaWeCo rejects any liability for personal injuries and material damages arising from this.

## 9. How to determine the correct breast shield size?

To express breast milk gently and efficiently, a vacuum has to develop within the breast shield.

### Correct fit of breast shield:

- Constant movement of the nipple corresponding to the pumping interval
- Comfortable feeling during pumping
- Evenly soft breast tissue after pumpin

### Breast shield too small or large:

- Slow milk flow and low milk volume
- Nipple rubs along side of the tunnel or Areola is pulled into the tunnel
- Uncomfortable feeling during pumping

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