# CBC Penlight LED

## Made in Germany

GB2070002M

Stand | As of | Édition 2023-05

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Available colours

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silber | silver | argent

REF 200250

blau | blue | bleu

REF 200260

rot | red | rouge

REF 200270

lila | purple | violet

REF 200280

limette | lime | lime

REF 200291

schwarz | black | noir

REF 200201

Kimetec GmbH  
Gerlinger Str. 36-38  
71254 Ditzingen  
GERMANY

[www.kimetec.de](http://www.kimetec.de)

## EN: Operating instructions

**Read these instructions carefully before using the CBC Penlight LED and keep them in a safe place for future reference.**

### **Intended use:**

The CBC Penlight LED is a medical device for examination of the pupillary reflex and for illumination of mouth and pharynx.

### **Contraindication/ side effect:**

None known.

### **User:**

This medical device should only be used by trained personnel.

### **Patient target group:**

No restrictions.

### **Switching on:**

Remove the grey transportation lock and press the push button until it clicks in.

### **Switching off:**

Press the push button again and it will return into its original position.

### **Battery equipment:**

2 x 1.5 V micro batteries LRO3 AAA. The use of rechargeable batteries is not permitted.

### **Changing batteries:**

Unscrew the bulb holder and remove the batteries. Insert new batteries with the positive pole showing to the bulb.

### **⚠ Safety notes: ⚠**

Do not use when allergic to nickel!

Check the product for any visible damage and functionality prior to each use. In the event of defects or malfunctions, discontinue use of the product.

### **Reprocessing:**

Disinfect the product prior to and after each use. Do not use alkaline detergents. Sterilization, mechanical disinfection, immersive disinfection and cleaning in an ultrasonic bath are not permitted.

### **Manual cleaning:**

Carefully wipe the CBC Penlight LED casing with a cloth moistened with a pH neutral detergent.

### **Wipe disinfection:**

Use alcohol-based disinfectants approved by the German Society of Hygiene and Microbiology DGHM.

### **Performance characteristics:**

Life time LED: ca. 100.000 h

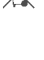
Luminous intensity: 15.000 – 31.000 mcd

Color temperature: ca. 3000 K

The lasered scale helps to determine the pupil width.

### **Ambient conditions for operation:**

 75%  
30% Humidity range

 106 kPa  
70 kPa Atmospheric pressure

 +40°C  
+10°C Ambient temperature

 Don't dispose of in domestic waste

---

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the national competent authority.

 Conformity mark according to Regulation (EU) 2017/745

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